AT03025, AT03025(MAX), AT03026, AT03026(MAX)

**INSTRUKCJA UŻYTKOWANIA
PODUSZKA ORTOPEDYCZNA Z OLEJKIEM RYCYNOWYM
USER MANUAL
ORTHOPEDIC PILLOW WITH CASTOR OIL
GEBRAUCHSANWEISUNG
ORTHOPÄDISCHES KISSEN MIT RIZINUSÖL
NÁVOD K POUŽITÍ
ORTOPEDICKÝ POLŠTÁŘ S RICINOVÝM OLEJEM
NÁVOD NA POUŽITIE
ORTOPEDICKÝ VANKÚŠ S RICÍNOVÝM OLEJOM
MODE D'EMPLOI
COUSSIN ORTHOPÉDIQUE À L'HUILE DE RICIN
GEBRUIKSAANWIJZING
ORTHOPEDISCH KUSSEN MET RICINUSOLIE
MANUAL DE INSTRUCCIONES
ALMOHADA ORTOPÉDICA CON ACEITE DE RICINO
ISTRUZIONI PER L'USO
CUSCINO ORTOPEDICO CON OLIO DI RICINO
BRUKSANVISNING
ORTOPEDISK KUDDE MED RICINOLJA**

PL

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EN

Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage or personal injury.

NOTE: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact your dealer for more information.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this product is intended (see the section on the purpose of the product of this manual). The product may be purchased by the user independently or on the recommendation of a doctor, therapist or other specialist. Whether purchasing the product on its own or as indicated by a doctor/therapist/other specialist, it is important to consider the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as the information provided by the manufacturer.

USE/INDICATIONS

The castor oil orthopedic pillow is a great choice for those looking for comfortable support for the cervical spine. Made of high-quality, high-density viscoelastic foam, it perfectly adapts to the shape of the body, guaranteeing proper support for the head and neck. Thanks to its shape memory function, it effectively reduces morning pain in the head, neck and shoulder area, while preventing problems resulting from an incorrect position during sleep. The specially contoured pillow provides comfort and support, especially for those who prefer sleeping on their side, improving the quality of sleep.

The orthopedic pillow provides support for the cervical spine to help maintain proper sleep position, and castor oil can promote muscle recovery and relieve pain due to its anti-inflammatory properties.

Castor oil is often used to relieve joint and muscle pain. When combined with an orthopedic pillow that helps maintain an anatomically correct position, it can promote pain reduction in the neck and shoulder areas.

Those struggling with morning headaches, neck or shoulder pain can benefit from the combination of an orthopedic pillow and castor oil. The pillow helps maintain proper posture, while the castor oil further relieves muscle tension.

By supporting the cervical spine, the orthopedic pillow helps minimize tension that leads to headaches. Castor oil applied topically to the neck can have a relaxing and anti-inflammatory effect, enhancing the effect.

Castor oil has anti-inflammatory and soothing properties, which can promote skin regeneration in the neck and shoulder area.

An orthopedic pillow helps relax neck muscles, and castor oil, can speed up tissue recovery and reduce neck stiffness.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling.

Although orthopedic pillows are designed to relieve muscle tension and discomfort in the neck, spine and shoulder areas, they can worsen symptoms in some people, especially at the beginning of use. The body needs time to adjust, but if the pain worsens or persists, it may indicate an improper choice of pillow and use should be discontinued.

People who have undergone surgery or have serious spinal injuries should consult a doctor before using an orthopedic pillow. An ill-fitting pillow can harm the healing and rehabilitation process.

Some people may find it difficult to get used to the change in pillow, especially if they have used traditional pillows for years. The adaptation period can last from a few days to a few weeks, and some people may experience discomfort during this time.

People who sleep on their stomachs may find it difficult to use an orthopedic pillow because it forces a position in which the head and neck are elevated, which can cause breathing discomfort and tension in the neck area.

Improper cleaning or lack of hygiene can lead to the proliferation of mites and other allergens. Not suitable for children under three years of age.

LIST OF COMPONENTS

Pillow, instruction manual

FEATURES

- Dense viscoelastic foam with memory effect
- Profiling under the head and neck
- Saturated with castor oil
- Prevents morning headaches, neck and shoulder pains
- Increases sleep comfort
- Covers made of knitted bamboo with navy blue cotton trim

DIMENSIONS

(length x width x height)

AT03025

-36 x 60 x 11[cm].

AT03025 MAX

-36 x 60 x 11[cm] including a 3 cm high bottom pad

AT03026

-30 x 60 x 12[cm].

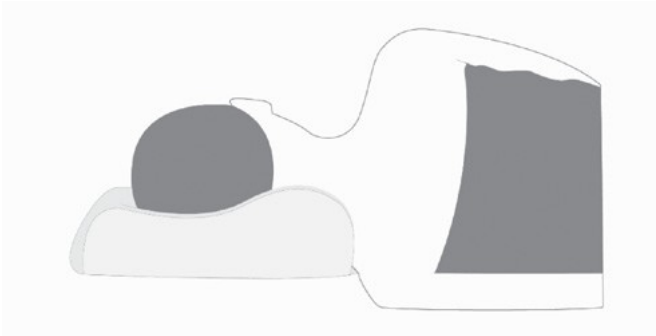
AT03026 MAX

-30 x 60 x 12[cm] including a 3 cm high bottom pad

Dimensions given may vary within +-2cm

USE

Arrange so that the higher back of the pillow is below the neck, as shown in the figure below:

MAX CUSHIONS

With the additional foam, the cushion can be elevated, customizing it to increase comfort. The whole unit consists of three components (pillow, 2-piece booster pad), which can be used simultaneously or use the pillow alone without the pad.

The booster pad consists of 2 parts that can be used together, or just one part of it, depending on your height preference.

NOTES:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person or
- (b) temporary or permanent deterioration of the health of the patient, user or other person or
- (c) a serious threat to public health

the above "serious incident" should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

If you experience pain, allergic reactions, or other disturbing symptoms that are unclear to you related to the use of the medical device, consult a healthcare professional.

CAUTION: it is forbidden to use the product other than for its intended purpose.

CLEANING AND CARE

Cleaning viscoelastic foam (also known as memory foam) requires a delicate approach to avoid damaging its structure. Here's a guide to properly clean it:

What You'll Need:

Mild detergent or dish soap

Warm water

Baking soda (optional for deodorizing)

Vacuum cleaner (preferably with a soft brush attachment)

Spray bottle

Soft cloth or sponge

Enzyme cleaner (optional for stains)

Steps for Cleaning:

Vacuum the Foam:

Remove dust and debris: Use a vacuum cleaner with a soft brush attachment to gently vacuum the surface of the foam. This will remove dust, dirt, and debris before you begin any wet cleaning.

Spot Clean with Mild Detergent:

Mix detergent: Combine a few drops of mild detergent with warm water in a spray bottle.

Lightly mist the foam: Avoid soaking the foam. Lightly spray the mixture on the affected area.

Blot the stains: Use a soft cloth or sponge to gently blot the area. Do not scrub, as this could damage the foam's structure.

Rinse the area: If needed, lightly spray with clean water to rinse off any soap residue.

Dealing with Tough Stains:

For urine or biological stains: Use an enzyme cleaner. Lightly apply the enzyme cleaner to the stain, let it sit for a while (following product instructions), then blot with a clean cloth.

Oil-based stains: Use a small amount of dish soap or a specialized upholstery cleaner designed for foam materials.

Deodorizing (Optional):

Use baking soda: Sprinkle a layer of baking soda on the foam and let it sit for a few hours, or even overnight, to absorb odors.

Vacuum off the baking soda after it has sat for long enough.

Dry the Foam:

Air dry completely: Foam is very absorbent, so it can take time to dry. Leave the foam in a well-ventilated area.

Use a fan to speed up the process: Never use direct heat, such as a hair dryer, as this can damage the foam. Just a cool fan is ideal.

Ensure the foam is fully dry before using it again to prevent mold or mildew growth.

What to Avoid:

Never submerge the foam in water, as it can be difficult to dry and may cause damage.

Avoid using harsh chemicals like bleach or alcohol-based cleaners that can degrade the foam.

No direct heat: Heat can damage the structure of the memory foam, so avoid putting it in the dryer or using a hot air source to dry.

CLEANING THE COVER: unzip the cover with the zipper and remove the foam. The cover can be hand washed at a maximum temperature of 40°C. Do not iron. Do not dry clean. Do not tumble dry.

	Hand wash at up to 40 degrees C
	Do not bleach
	Do not iron
	Do not tumble dry
	Do not dry clean

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight. Not suitable for children under three years of age. Store away from sources of fire.

UTILIZATION

Once the device is decommissioned, the medical device can be disposed of as ordinary municipal waste.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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