AT03028

**INSTRUKCJA UŻYTKOWNIA, PODKŁADKA OCHRONNA POD PIĘTĘ
USER MANUAL, PROTECTIVE PAD UNDER THE HEEL
BENUTZERHANDBUCH, SCHUTZPOLSTER UNTER DER FERSE
NÁVOD K POUŽITÍ, OCHRANNÁ PODLOŽKA POD PATU
POUŽÍVATEĽSKÁ PRÍRUČKA, OCHRANNÁ PODLOŽKA POD PÄTU
MANUEL D'UTILISATION, COUSSIN DE PROTECTION SOUS LE TALON
HANDLEIDING, BESCHERMKUSSEN ONDER DE HIEL
MANUAL DE USUARIO, ALMOHADILLA PROTECTORA BAJO EL TALÓN
MANUALE D'USO, CUSCINETTO PROTETTIVO SOTTO IL TALLONE
ANVÄNDARMANUAL, SKYDDSDYNA UNDER HÄLEN**

MD

CE

PL

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EN

Thank you for purchasing our product. Please read the user manual carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, notes, and recommendations, contact a healthcare specialist or the seller to avoid product damage or bodily injury.

WARNING: Check all parts for damage that may have occurred during transport. If such damage is found, DO NOT use the product. For more information, contact the seller.

TARGET PATIENT GROUP

Individuals suffering from diseases, dysfunctions, or injuries for whose treatment, rehabilitation, or compensation this product is intended (see the section on product purpose in this manual). The product can be purchased independently by the user or based on recommendations from a doctor, therapist, or other specialist. Whether purchasing the product independently or based on a specialist's recommendation, one should consider the available sizes, necessary functions, product variants, indications, contraindications, and information provided by the manufacturer.

APPLICATION / INDICATIONS

The protective heel pad is made of soft polyurethane foam and is intended for individuals at risk of developing pressure ulcers on the heels. The pad has a contoured inner surface that ensures good air circulation and reduces pressure on soft tissues. It is equipped with Velcro fasteners for better stabilization.

The pad protects the foot from abrasions in the heel and ankle areas. Additionally, it supports pressure ulcer prevention and provides free air circulation.

Using the product reduces the risk of pressure ulcers, but does not eliminate it entirely. To minimize the risk of pressure ulcers, regardless of product use, proper patient care is essential, particularly regular repositioning of the patient.

CONTRAINDICATIONS

Physical or mental limitations (e.g., vision impairment) that prevent safe handling of the product.

Skin damage: The product is not intended for use in cases of open wounds, ulcers, skin infections, burns, or other skin injuries in the heel or ankle area.

Circulatory problems: Individuals with severe circulatory issues, such as thrombosis or venous insufficiency, should consult a doctor before using the pad to ensure it does not worsen their condition.

Dermatological conditions: People with skin diseases such as eczema, psoriasis, or other chronic skin issues should consult a dermatologist before using the pad.

Sensory disorders: Individuals with sensory impairments in the feet (e.g., neuropathy) should consult a doctor before using the product to avoid improper fitting that could lead to further injuries.

LIST OF COMPONENTS

Two protective heel pads, User manual

FEATURES

Made of polyurethane foam

Velcro fasteners

Universal size: 20 x 17.5 cm

USAGE INSTRUCTIONS

Ensure the pad is clean and dry before use. If necessary, gently wipe it with a damp cloth.

Place the pad on the heel so that the contoured inner surface fits snugly around the skin of the heel and ankle. Ensure that the pad is neither too loose nor too tight to maintain comfort and effectiveness.

After positioning the pad on the heel, fasten the Velcro straps for proper stabilization. Make sure the pad stays securely on the foot but is not too tight to avoid pressure. Adjust the Velcro straps so the pad does not shift while walking.

Breaks in usage:

If you experience any discomfort or skin irritation, remove the pad immediately. Discontinue use in case of any abnormalities and consult a doctor, especially if you feel pain or notice redness around the heels.

WARNING:

In the event of a serious incident related to the product that directly or indirectly resulted in, could have resulted in, or may result in any of the following:

- a) death of the patient, user, or another person;
- b) temporary or permanent deterioration of the health of the patient, user, or another person;
- c) a serious public health threat,

this serious incident must be reported to the manufacturer and the competent authority of the EU member state in which the user or patient resides.

WARNING:

If you experience pain, allergic reactions, or any other concerning or unclear symptoms related to the use of the medical device, consult a healthcare professional.

WARNING:

The product must not be used in any manner other than its intended purpose.

CLEANING

WARNING: DO NOT MACHINE WASH!

Cleaning foam requires a gentle approach to avoid damaging its structure. Here is a guide on how to clean it properly:

What You Will Need:

Mild detergent or dish soap

Warm water

Baking soda (optional for deodorization)

Vacuum cleaner (preferably with a soft brush attachment)

Spray bottle

Soft cloth or sponge

Enzyme cleaner (optional for stain removal)

Cleaning Steps:

Vacuuming the Foam:

Remove dust and debris: Use a vacuum cleaner with a soft brush attachment to gently clean the foam surface. This removes dust, dirt, and debris before proceeding with wet cleaning.

Spot Cleaning with Mild Detergent:

Mix the detergent: Combine a few drops of mild detergent with warm water in a spray bottle.

Lightly spray the foam: Avoid soaking the foam. Lightly mist the stained area with the solution.

Blot the stains: Use a soft cloth or sponge to gently blot the area. Do not rub, as this may damage the foam structure.

Rinse the area: If needed, lightly spray with clean water to remove any detergent residue.

Handling Tough Stains:

Biological stains or urine: Use an enzyme cleaner. Apply a small amount to the stain, let it sit for a few minutes (as per product instructions), then blot with a clean cloth.

Greasy stains: Use a small amount of dish soap or a specialized upholstery cleaner suitable for foam materials.

Deodorization (Optional):

Use baking soda: Sprinkle a layer of baking soda over the foam and let it sit for a few hours or overnight to absorb odors.

Vacuum the baking soda after sufficient time has passed.

Drying the Foam:

Air dry completely: Foam is highly absorbent, so drying may take time. Leave it in a well-ventilated area.

Use a fan to speed up drying: Never use direct heat sources, such as a hairdryer, as they can damage the foam. A cool air fan works best.

Ensure the foam is completely dry before reuse to prevent mold or mildew growth.

What to Avoid:

Never submerge the foam in water, as it may be difficult to dry and could become damaged.

Avoid harsh chemicals such as bleach or alcohol-based cleaners, which can weaken the foam structure.

No direct heat: Heat can damage the foam, so avoid tumble drying or using hot air.

STORAGE

Store the product in a dry place and keep it away from direct sunlight.

DISPOSAL

Once the product is no longer in use, dispose of it as regular household waste. Follow local waste segregation guidelines to protect the environment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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