AT51011/AT51012
INSTRUKCJA UŻYTKOWNIA, KRZESŁO PRYSZNICOWE
INSTRUCTION MANUAL, SHOWER CHAIR
GEBRAUCHSANWEISUNG, DUSCHSTUHL
NÁVOD K POUŽITÍ, SPRCHOVÁ ŽIDLE
NÁVOD NA POUŽITIE, SPRCHOVÁ STOLIČKA
NOTICE D'UTILISATION, CHAISE DE DOUCHE
GEBRUIKSAANWIJZING, DOUCHESTOEL
MANUAL DE USO, SILLA DE DUCHA
ISTRUZIONI PER L'USO, SGABELLO/SEDIA DA DOCCIA
BRUKSANVISNING, DUSCHSTOL

PL

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EN

Thank you for purchasing our product. Please read this user manual carefully. Do not use the product before reading and understanding this instruction manual. If you do not understand the warnings, notes, and recommendations, please consult a healthcare professional or the seller to avoid damage to the product or bodily injury.

NOTE: Check all parts for any damage that may have occurred during transport. If such damage is noticed, DO NOT use the product. For more information, please contact the seller.

TARGET PATIENT GROUP

Individuals suffering from diseases, dysfunctions, or injuries for which this product is intended for treatment, rehabilitation, or compensation (see section on intended use in this manual). The product may be purchased independently by the user or on the recommendation of a doctor, therapist, or other specialist. Whether purchased independently or prescribed, the available sizes, required functions, variants, indications, contraindications, and information provided by the manufacturer must be considered.

APPLICATION

Shower chairs for people with disabilities are designed for individuals with disabilities or limited mobility. They enable or assist in safe and comfortable use of the shower. The product facilitates hygiene activities for persons who: have reduced mobility, experience problems with balance or stability, have difficulty standing for long periods, are in recovery after injuries or surgeries, are elderly and require additional support while bathing. The chair may be used both at home and in care facilities, hospitals, and nursing homes.

INDICATIONS

Use of the shower chair is recommended for: weakened muscles or joints, neurological or orthopedic diseases affecting mobility, dizziness or fainting tendencies, balance disorders (e.g., Parkinson's disease, multiple sclerosis), postoperative periods when prolonged standing is not advised, overweight individuals (taking into account the maximum load of 136 kg).

CONTRAINDICATIONS

Use of the shower chair is not recommended in the following cases: by persons exceeding the maximum allowable weight (136 kg), in shower cabins or bathtubs where stable placement of all chair legs cannot be ensured, when structural parts are damaged (cracks, bends, missing mounting elements), by persons with severe disorders of consciousness or behavior that prevent safe use, by children without constant adult supervision, physical or mental limitations (e.g., impaired vision) that prevent safe handling of the product.

COMPONENTS: Seat – 1 pc. Height-adjustable legs with rubber, non-slip feet – 4 pcs. User manual. Product AT51012 additionally includes: Backrest – 1 pc.

FEATURES

Ergonomic shape adapted to the user's body.
Double "U"-shaped seat with drainage holes for better hygiene.
Five-level height adjustment to suit the user's height and needs.
Rubber non-slip feet ensuring stability on wet surfaces.
Easy assembly and disassembly without tools.
Cut-out for shower head placement.
AT51012: includes backrest

SPECIFICATION

Material: PP plastic, aluminum, rubber
Maximum load: 136 kg
Height adjustment range: 5 levels

DIMENSIONS AT51011

Overall width: 55 cm
Overall height: 42–55 cm (adjustable)
Seat width: 55 cm
Seat depth: 41 cm
U-shaped cut-out depth: 17 cm
Weight: 2 kg

DIMENSIONS AT51012

Overall width: 55 cm
Overall height: 80–91 cm (adjustable)
Seat height from the ground: 42–55 cm (adjustable)
Seat width: 55 cm
Seat depth: 41 cm
U-shaped cut-out depth: 17 cm
Backrest height: 19 cm
Weight: 3 kg

NOTE: Individual product dimensions may vary within ± 2 cm.

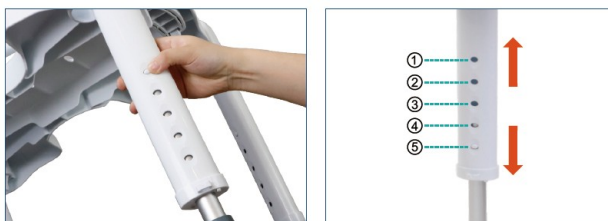
This symbol indicates the maximum user weight.

ASSEMBLY

Remove the seat, turn it upside down. Insert the four legs into the appropriate sockets and twist to secure.

Adjust the legs to the same height (five levels available).

Insert the backrest into the two rear sockets of the seat.

WARNINGS AND SAFETY

If a “serious incident” related to the product occurs that directly or indirectly led, could have led, or may lead to:

- a) death of the patient, user, or another person,
- b) temporary or permanent impairment of health,
- c) serious public health threat,

this “serious incident” must be reported to the manufacturer and the competent authority of the EU member state where the user or patient resides. In case of pain, allergic reactions, or other concerning symptoms while using the product, consult a healthcare professional. Do not use the product in any way other than its intended purpose.

ADDITIONAL SAFETY WARNINGS:

Use only as intended – the chair is designed exclusively for sitting while showering. Do not exceed the maximum load – 136 kg. Place on a stable, even surface – ensure all legs are firmly placed before use. Always check stability before sitting – ensure the chair is correctly assembled and tightened. Be cautious when getting on or off – use stable bathroom handles or rails. Do not use if damaged – cracks, deformations, or worn elements may cause falls. Do not modify the structure – any alterations may reduce safety. Use under supervision if the user is at risk of losing balance, unconsciousness, or fainting. Regularly inspect the adjustment mechanisms – ensure all legs are the same height and secure.

CLEANING AND MAINTENANCE

After each use, rinse the chair with clean water to remove soap, shampoo, and other residues. Clean using a soft cloth or sponge with mild detergent. Do not use harsh chemicals, solvents, or abrasive cleaners. After cleaning, dry thoroughly with a towel or leave in a ventilated place to air dry completely. Regularly inspect all components, especially rubber feet, leg attachments, armrests, and backrest. Tighten any loose screws or fasteners. If cracks, deformations, or other damage are noticed, discontinue use until repaired or replaced.

STORAGE: Store in a dry place. Do not expose to direct sunlight.

DISPOSAL: When the product is no longer in use, dispose of it as regular household waste. Observe waste segregation regulations. Care for the environment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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