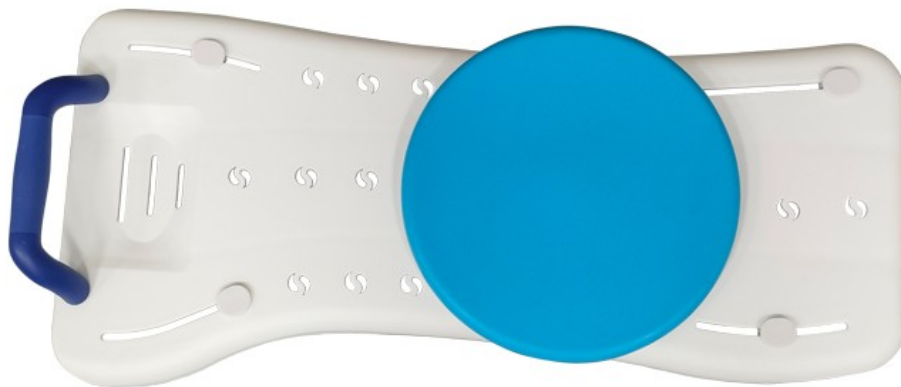
AT51053
INSTRUKCJA UŻYTKOWNIA, ŁAWKA NAWANNOWA
USER MANUAL, BATH BENCH
GEBRAUCHSANWEISUNG, BADEBANK
NÁVOD K POUŽITÍ, SEDÁTKO DO VANY
NÁVOD NA POUŽITIE, LAVICA DO VANE
NOTICE D'UTILISATION, SIÈGE DE BAIN
GEBRUIKSAANWIJZING, BADBANK
MANUAL DE INSTRUCCIONES, ASIENTO DE BAÑERA
MANUALE D'USO, PANCA DA BAGNO
BRUKSANVISNING, BADBÄNK

PL

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Thank you for purchasing our product. Please read this manual carefully before use. Do not operate the device before reading and fully understanding these instructions. If you do not understand the warnings, precautions, or recommendations, contact a Healthcare professional or the seller to avoid product damage or personal injury. **WARNING** Check all parts for any damage that may have occurred during transport. If any damage is found, DO NOT use the product. For more information, contact your seller

INTENDED USER GROUP

This product is intended for individuals suffering from diseases, dysfunctions, or injuries for which this device is designed for treatment, rehabilitation, or compensation (see the “Intended Use” section of this manual).

The product may be purchased directly by the user or based on recommendations from a physician, therapist, or another specialist.

In both cases (self-purchase or medical recommendation), the following must be considered:

- available sizes, required functions, and product variants,
- indications and contraindications for use,
- and information provided by the manufacturer.

INTENDED USE: The bath bench is designed for people with disabilities or reduced mobility to enable safe and comfortable bathing.

INDICATIONS

Persons with limited mobility who cannot safely use a traditional bathtub. Elderly individuals requiring additional support and stability during bathing. Patients recovering from injuries or surgeries of the lower limbs, hips, or spine. Persons with neurological or orthopedic conditions causing balance disorders or difficulty standing for long periods. Overweight persons (within the product’s maximum load limit).

CONTRAINDICATIONS

Physical or mental impairments (e.g., visual impairment) preventing safe product handling. User weight exceeding the maximum permissible load (see product specifications). Bathtubs with non-standard shapes or slippery edges preventing stable mounting of the bench. Visible structural damage (cracks, bends, worn anti-slip elements). Persons with severe consciousness or behavioral disorders preventing safe use. Children without adult supervision. Do not use the bench in a standing position or as a step. The product is intended for sitting only.

SAFETY PRECAUTIONS

Always ensure that the bench is properly and securely mounted on the bathtub edges. Check the condition of anti-slip and fastening elements before each use. Do not modify the structure yourself and use the product only as intended. If there is a risk of loss of balance, fainting, or neurological issues, use the bench under supervision. After each use, rinse the bench with clean water and dry it to prevent soap residue buildup and reduced grip.

WARNING: The bench has no back support. The product must not be used by patients with balance disorders or by those who, for various reasons, are unable to sit independently. Such patients may fall backward and hit the edge of the bathtub, which can result in serious injuries.

LIST OF COMPONENTS: 1× bathtub board, 1× rotating seat, 1× support handle, 3× round plastic screws, 4× rectangular plastic screws, 4× board feet, 4× plastic washers, 1× user manual

TECHNICAL SPECIFICATION

Weight: 2.65 kg

Total length: 75.5 cm

Total height: 18 cm

Seat height: 4.5 cm

Seat width: 28 cm

Diameter of rotating seat: 30 cm

Usable surface length (between handles): 63 cm

Material: PP (polypropylene), TPR (thermoplastic rubber / elastomer)

Maximum user weight: 100 kg

This symbol indicates the maximum user weight.

Note: The individual product dimensions may vary by ± 2 cm.

ASSEMBLY INSTRUCTIONS

Step 1: Installing the handle

Insert the handle into the two holes at the end of the bathtub board.

For user comfort, the handle can be positioned facing inward or outward.

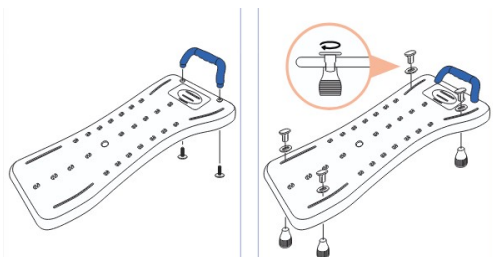
Insert the screws on both sides of the handle and tighten until stable.

Step 2: Installing the adjustable feet

Using the rectangular plastic screws and washers, attach the four feet to the underside of the board.

The screw passes through the surface, then the washer underneath, and finally into the foot.

Tighten until secure.

IMPORTANT NOTICE

In the event of a serious incident related to the product that has led, might have led, or could lead to any of the following:

- a) death of a patient, user, or another person;
- b) temporary or permanent deterioration of health of a patient, user, or another person;
- c) serious threat to public health –

such a serious incident must be reported to the manufacturer and to the competent authority of the Member State where the user or patient resides.

In the United Kingdom, the competent authority is the Medicines and Healthcare products Regulatory Agency (MHRA).

ADDITIONAL WARNING

If pain, allergic reactions, or other concerning or unclear symptoms occur while using the medical device, consult a healthcare professional.

It is forbidden to use the product in any way other than its intended purpose.

CLEANING

After each use, rinse the product with clean water to remove soap, shampoo, and other residues.

Use a soft cloth or sponge with a mild detergent.

Do not use harsh chemicals, solvents, or abrasive cleaners, which may damage the surface.

After washing, dry the product or leave it in a well-ventilated area until completely dry.

Regularly check the condition of all components – especially fixings and screws.

If any become loose, tighten them.

If cracks, deformations, or other structural damage are observed, stop using the product until repaired or replaced.

STORAGE

Store in a dry place, away from direct sunlight.

DISPOSAL

After the end of service life, dispose of the product as household waste.

Follow local waste segregation regulations.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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