AT52327

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En

Thank you for purchasing our product.

Please read the user manual carefully before using the product. Do not operate the device before fully understanding this manual. If you do not understand the warnings, precautions, or instructions, contact a healthcare professional or the seller to avoid product damage or personal injury.

ATTENTION: Check all parts for damage that may have occurred during transport. If any damage is detected, **DO NOT USE** the product. For more information, contact the seller.

TARGET USER GROUP

This product is intended for individuals experiencing medical conditions, dysfunctions, or injuries requiring treatment, rehabilitation, or mobility compensation (see the section on product purpose in this manual). The product may be purchased by the user independently or upon the recommendation of a doctor, therapist, or other specialist.

Regardless of whether the product is purchased independently or upon a specialist's recommendation, it is essential to consider the **available sizes, necessary functions, product variants, indications, contraindications for use, and information provided by the manufacturer.**

GENERAL SAFETY PRECAUTIONS

The powered wheelchair is designed for both indoor and outdoor use.

Use extreme caution when driving on unknown terrain or soft ground.

Always ensure that the wheelchair is fully unfolded before sitting down. Do not fold the wheelchair until you have completely exited, the backrest is folded down, and the armrests are in their lowest position.

Do not operate the adjustment mechanism while someone is seated.

Read and follow the instructions in the user manual. Do not use this product or any available optional equipment without thoroughly reading and understanding these instructions.

If you do not understand the warnings, precautions, or instructions, contact the seller before using the device. Failure to do so may result in injury or product damage.

Never use the powered wheelchair beyond its specified limitations as described in this manual.

Always keep your feet on the footrest while driving. Do not stand on the footrest.

Operate only on flat surfaces where all four wheels maintain contact with the ground to ensure safe wheelchair operation.

If the battery gauge drops to red, recharge the wheelchair as soon as possible. Fully discharging the battery may damage it or shorten its lifespan.

Keep metal objects away from the battery terminals – electric shock may occur.

Always reduce speed and maintain a stable center of gravity when turning. Do not make sharp turns at high speeds, and always slow down before turning.

Check for obstacles behind you before reversing.

Do not allow children to play near or operate the powered wheelchair.

Some wheelchair components may be sensitive to temperature changes. The controller operates correctly within a temperature range of -10°C to 50°C (-14°F to 122°F).

In extremely low temperatures, batteries may freeze, making the wheelchair inoperable. At high temperatures, the wheelchair may move more slowly due to a built-in safety function that prevents damage to the motors and other electrical components.

Store the wheelchair in a clean and dry place.

Do not use the wheelchair at night without proper lighting.

Do not remove the anti-tip wheels or transit wheels.

Keep hands and feet away from moving parts while driving. Be cautious of loose clothing that may get caught in the drive wheels.

The surface temperature of the wheelchair may increase when exposed to external heat sources.

Do not connect an extension cord to the battery charger.

Disassembling the controller, motor, or charger by anyone other than an authorized service agent is prohibited and voids any applicable warranty.

Do not operate the powered wheelchair on roads or highways.

Do not use the wheelchair if under the influence of alcohol, medications, or other substances that may impair driving abilities.

Exercise caution when operating the wheelchair in crowded areas or shopping malls.

Under no circumstances should the powered wheelchair be used as a seat in a motor vehicle.

Do not lift the wheelchair by any parts other than the frame (e.g., seat or body panels).

APPLICATION / INDICATIONS

The powered wheelchair is designed to assist in the transport of individuals. It is intended for use by **one person** with a **maximum weight of 136 kg** and is **not designed for carrying passengers (including children)**. The powered wheelchair is **for adults only** and **not for transporting goods or individuals under 16 years of age**. It is a **mobility assistance device** designed to **compensate for movement difficulties**.

Severe mobility impairments – Individuals with **significant muscle weakness**, such as those with **neurological or muscular disorders**, or **chronic pain syndromes** that severely restrict independent movement.

Neurodegenerative diseases – Conditions such as **multiple sclerosis, Parkinson's disease, muscular dystrophy, or ALS**, which progressively impair muscle control and strength.

Fatigue-related mobility restrictions – Individuals who **tire quickly due to medical conditions** and **cannot manually operate a wheelchair for extended periods**.

Age-related mobility limitations – Elderly individuals with **reduced mobility and strength** who require assistance for independence and movement.

Rehabilitation and recovery – Individuals undergoing **rehabilitation after injuries or surgeries**, who **temporarily** cannot move independently, can use a powered wheelchair as part of the healing process.

CONTRAINDICATIONS

Physical or cognitive impairments (e.g., visual impairment) that prevent safe operation of the wheelchair.

Severe difficulties in judging distances or spatial awareness, which may affect safe maneuvering.

Lack of manual dexterity or coordination required for operating the joystick or other control devices.

Severe cognitive disorders that impair the ability to safely control the wheelchair.

Individuals unable to maintain a stable seated position may require additional support systems to use the wheelchair safely.

Epilepsy or other conditions that may cause sudden loss of awareness or delayed reaction times.

Severe motor coordination difficulties that prevent safe control of the wheelchair.

Consciousness disorders that impair safe vehicle operation.

Certain cardiovascular conditions or blood pressure disorders affect safe wheelchair use that could

Progressive neurological diseases (e.g., advanced multiple sclerosis) that require specialized adaptive solutions.

Depth perception and spatial orientation problems, which may limit safe operation.

General poor health or recent surgeries (especially involving the spine or lower limbs), which may require temporary suspension of wheelchair use.

LIST OF INCLUDED COMPONENTS

Powered wheelchair, User manual

TRANSPORTING THE WHEELCHAIR

When transporting the electric wheelchair in a vehicle, it should be securely placed in the rear section of a van or truck. Adjustable parts should be removed or properly secured during transport.

The electric wheelchair is suitable for both land and sea transport.

To ensure user safety and avoid damage to the wheelchair during transport, securing straps must be attached to the designated securing points (A) and (B) on the wheelchair, following the instructions provided by the restraint system supplier.

INCLINE GUIDELINES

Exercise caution when approaching inclines or declines; if necessary, reduce speed before entering a slope.

While driving uphill, try to keep the electric wheelchair moving forward. If you must stop, proceed with caution and accelerate slowly to continue moving.

When descending a slope, reduce the wheelchair speed to the lowest setting and proceed carefully. If the wheelchair moves faster than expected, gradually release the joystick to stop and then gently push it forward to continue descending safely.

WARNING – Always drive the wheelchair straight up or down an incline, never in reverse. Sudden movements or forward-backward driving patterns can increase the risk of tipping over. Changes in incline can reduce stability. Under no circumstances should you travel on an incline steeper than the maximum stability angle specified for this wheelchair.

OUTDOOR USE AND PRECAUTIONS IN ADVERSE WEATHER CONDITIONS

Although the electric wheelchair is designed for both indoor and outdoor use to support daily mobility, the manufacturer understands that outdoor use may sometimes be necessary and has designed the wheelchair to perform well on even outdoor surfaces.

CAUTION – When using the wheelchair outdoors, avoid uneven terrain, soft surfaces, tall grass, loose gravel, loose sand, fresh or salt water, as well as the edges of streams, lakes, and oceans. If you are uncertain about a surface, avoid it.

Exposure to extreme weather conditions should always be avoided. If you find yourself in poor weather conditions while using the wheelchair, immediately seek the nearest shelter. Before resuming use, charging, or storing the wheelchair, ensure that it is completely dry.

PHYSICAL LIMITATIONS

The electric wheelchair is designed and manufactured exclusively for transporting a single person with a maximum weight of 136 kg. The user should have good vision and maintained awareness. Caution, sound judgment, and common sense should always be exercised while using the wheelchair. Be aware of your own limitations and the effects of any substances that may impair your ability to operate the wheelchair.

WARNING – Never operate the electric wheelchair under the influence of alcohol or drugs. Do not smoke while sitting in the wheelchair. Damaged or worn upholstery increases the risk of fire and should be replaced immediately.

CAUTION – Be aware of precautions, warnings, and safety considerations when taking prescription or over-the-counter medications while using the electric wheelchair.

ELECTROMAGNETIC INTERFERENCE (EMI) INFORMATION

Electromagnetic interference (EMI) testing has shown that it may negatively affect the operation and control of electrically powered mobility devices.

EMI can be generated by various sources, including mobile phones, two-way radios, radio and television stations, HAM radio transmitters, wireless computer connections, microwave signals, paging transmitters, and transmitters/receivers used by emergency vehicles.

EMI waves can cause unintended movement of the electric wheelchair or damage to the controller. Every electrically powered mobility device has a certain degree of resistance to EMI. The higher the resistance level, the greater the protection. The intensity of interference can be measured in volts per meter (V/m). Current technology allows for a resistance level of 20 V/m, which provides protection against most common EMI sources. Your electric wheelchair, without any modifications, has a resistance level of 20 V/m.

EMI interference becomes stronger as the source gets closer. Portable radios pose a particular risk. The wheelchair's control system may be unintentionally exposed to high levels of EMI, which can affect movement and braking. The following warnings are intended to help prevent potential interference with the wheelchair's control system.

WARNING – Do not use portable two-way radios, such as CB (citizens band) radios, and do not activate communication devices, such as mobile phones, when the electric wheelchair is turned on.

WARNING – Be cautious of nearby transmitters, such as radio or television stations, and avoid close proximity to them.

TECHNICAL SPECIFICATION

Maximum user weight: 136 kg

Maximum speed (forward motion): 6.4 mph (10 km/h)

Minimum braking distance at maximum speed: 100 cm – 170 cm

Range on full charge:

Lead-acid battery 24V52Ah: 30 km

Maximum obstacle height: 5 cm

Maximum safe incline: 10°

Minimum turning radius: 90 cm

Minimum width required for reversing: 90 cm

Folded dimensions (L x W x H): 85 x 60 x 97 cm

Dimensions when unfolded (L x W x H): 108 x 58 x 121.5 cm

Packaging dimensions (carton): 102 x 72.5 x 85 cm

Total weight (including battery): 106.5 kg (with 24V52Ah battery)

Battery weight: 30 kg

Seat and Footrest Dimensions

Effective seat depth: 43 – 49 cm

Effective seat width: 43 – 48 cm

Height of the front seat edge: 45 cm

Height from floor to seat (measured at the front): 45 cm

Armrest height: 17~29 cm

Backrest height: 43 cm

Footrest height: 13 – 20 cm

Seat surface inclination angle: 0 – 15° / 0 – 20°

Motor and Battery

Motor: 2x340 W, electromagnetic brake

Battery: 24V52Ah (C20)

Charger:

Input: 100-240V AC, 50-60Hz, 1.2-0.5A, Class II

Output: 24V DC, 6A

Maximum motor noise level: 68 dB

EMC compliance: ISO 7176-21

Upholstery flammability: ISO 16840-10

Wheels and Suspension

Rear Wheel : 14 inches

Front wheels: 9 inches

Suspension: Rear suspension

Storage and Usage Conditions

Storage and operating temperature: -10°C to +50°C

Storage and operating humidity: 30% – 70%

This symbol indicates the maximum user weight.

The given dimensions may vary within a tolerance of ± 2 cm.


ELEMENTS OF A WHEELCHAIR

1. Backrest
2. Armrest
3. Control console
4. Seat
5. Footrest
6. Footplate
7. Front wheels
8. Battery housing
9. Drive motors
10. Rear wheels / Drive wheels
11. Front lights
12. Rear lights
13. Seatbelt
14. Anti-tip wheels

CONTROL ELEMENTS OF AN ELECTRIC WHEELCHAIR

1. Battery charge indicator
2. Power button - Status indicator
3. Position lights
4. Horn
5. Right turn signal
6. Joystick
7. Left turn signal
8. Hazard lights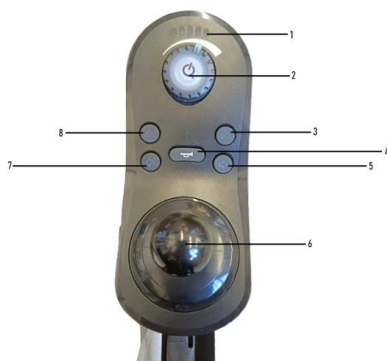

BATTERY CHARGE INDICATOR

When the power button is pressed to turn on the wheelchair, this indicator displays the approximate battery charge level.

POWER BUTTON / STATUS INDICATOR

To turn on the electric wheelchair, press the power button. This is the only operational button that activates the wheelchair system.

If the system is functioning properly, the status indicator (located on the power button) will light up green, and the battery indicator will show the current charge level.

If a system error occurs during startup, the status indicator will signal a malfunction with a series of red flashes. In such a case, contact a specialized dealer who will be happy to assist you in resolving the issue.

To turn off the electric wheelchair, press the power button again. The system will shut down, and the status indicator will turn off.

HAZARD LIGHTS

To turn the hazard lights on or off, press the "hazard lights" button.

TURN SIGNALS

To activate a turn signal, press the corresponding turn signal button (left or right).

POSITION LIGHTS

To turn on the position lights, press the "position lights" button.

SPEED CONTROL DIAL

The speed control dial allows the user to limit the wheelchair's maximum speed (i.e., the speed reached when the joystick is fully deflected), adjusting it to personal preferences and environmental conditions.

The dial offers 10 speed settings – from the lowest (dial turned to the left) to the highest (dial turned to the right).

For easier orientation, a speed symbol is located next to the dial, indicating the minimum and maximum speed positions.

HORN BUTTON

This button activates the warning signal.

The wheelchair must be turned on for the horn to work.

Press the horn button to activate the sound signal.

The horn will remain active as long as the button is pressed.

Do not hesitate to use the warning signal if it can help prevent an accident or injury.

JOYSTICK

The joystick controls the direction and speed of movement.

When the joystick is moved from its neutral position, the wheelchair will move in the same direction the joystick is pushed.

ANTI-TIP WHEELS

The anti-tip wheels on this electric wheelchair provide a high level of safety and stability.

Their special design enhances the wheelchair's stability, reducing the risk of tipping over while driving uphill.


MANUAL FREEWHEEL LEVER

If you ever need to move your electric wheelchair over short distances manually, you can switch it to freewheel mode.

Locate the manual freewheel lever at the front of the drive motors.

Push the freewheel lever inward to disable the drive system and braking system. You can now push your electric wheelchair manually.

Push the freewheel lever outward to re-engage the drive and braking systems – this will disable the wheelchair's freewheel mode.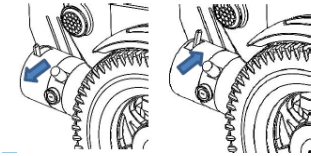

WARNING

Before switching the wheelchair into or out of freewheel mode, turn off the power.

Never sit in the wheelchair while it is in freewheel mode.

Never activate freewheel mode on slopes – in this mode, the braking system is disabled.

Only disengage the drive motors on a flat surface.

Ensure the wheelchair is turned off.

Stand beside the wheelchair when switching to or from freewheel mode.

Never do this while sitting in the wheelchair.

After moving the wheelchair, always restore drive mode to engage the brakes.

BEFORE GETTING ON THE ELECTRIC WHEELCHAIR

Are the batteries fully charged?

Is the manual freewheel lever in the drive position (pushed outward)? Never leave the lever in the inward position unless you are manually pushing the wheelchair.

Check the environmental conditions:

Air temperature: from -25°C (-13°F) to 50°C (122°F).

Air humidity: 30% – 70%.

Atmospheric pressure: 86 kPa – 106 kPa.

GETTING ON THE ELECTRIC WHEELCHAIR

Ensure the wheelchair is turned off.

WARNING – Never get on or off the wheelchair without turning off the power first. This prevents accidental movement if the joystick is touched.

Stand next to the wheelchair.

Unlock the footrest lever and rotate the footrests outward to allow easy access to the seat.

Sit comfortably and securely on the seat.

Rotate the footrests inward so your feet can rest on them comfortably.

Make sure your feet are securely positioned on the footrests.

Fasten the seatbelt.

ADJUSTMENT AND CHECKS BEFORE DRIVING

Is the seat in the correct position?

Is the seat securely locked?

Has the power button been pressed, and is the status indicator green?

Is the horn functioning properly?

Is your route clear of people, animals, and obstacles?

Have you planned your route to avoid difficult terrain and steep inclines?

OPERATING THE ELECTRIC WHEELCHAIR

WARNING – The following actions can negatively affect wheelchair control and stability:

Holding a leash to walk a pet while driving.

Carrying passengers (including animals).

Hanging any objects on the armrests or control console.

Towing or being pulled by another motor vehicle.

WARNING – Always keep both hands on the armrests and both feet on the footrests while driving. This position provides the best control over the wheelchair.

Set the speed control dial to the desired level.

Gently push the joystick in the desired direction.

The electromagnetic brakes will automatically release, and the wheelchair will accelerate smoothly to the pre-set speed.

To turn left, push the joystick toward the front left.

To turn right, push the joystick toward the front right.

To drive straight forward, push the joystick straight ahead.

To stop, slowly release the joystick. The electronic brakes will engage automatically when the wheelchair comes to a complete stop.

NOTE: The reverse speed is lower than the forward speed set by the speed control dial.

GETTING OFF THE ELECTRIC WHEELCHAIR

Bring the wheelchair to a complete stop.

WARNING – Never get on or off the wheelchair without turning off the power first. This prevents accidental movement if the joystick is accidentally touched.

Press the power button to turn off the wheelchair.

Unfasten the seatbelt.

Unlock the footrest lever and rotate both footrests outward to allow easy exit from the wheelchair.

Carefully stand up from the seat and move to the side of the wheelchair.

Rotate both footrests inward to return them to their original position.


ADJUSTING THE CONTROL CONSOLE

Turn the screw counterclockwise.

Loosen the controller adjustment tube.

Move the controller adjustment tube forward or backward to set the console to the preferred position.

Turn the screw clockwise to securely tighten the controller adjustment tube.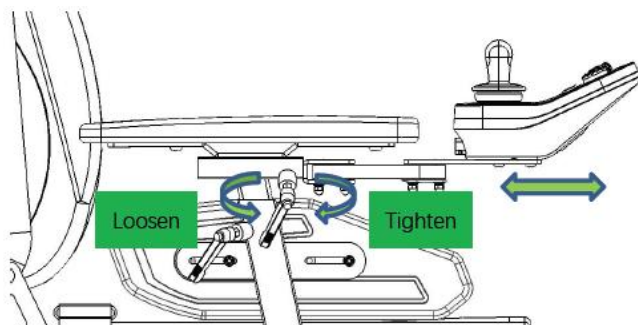

ADJUSTING THE HEIGHT AND WIDTH OF THE ARMRESTS

Adjusting the Width of the Armrests

1. Using the tools provided with the wheelchair, turn the two screws counterclockwise.


2. Loosen the armrest width adjustment tube.

3. Move the armrest outward or inward to adjust it according to your needs.

4. Repeat steps 1–3 for the armrest on the other side.

5. Check that both armrests are in the desired position.

6. Turn the screws clockwise to tighten the armrest width adjustment tubes.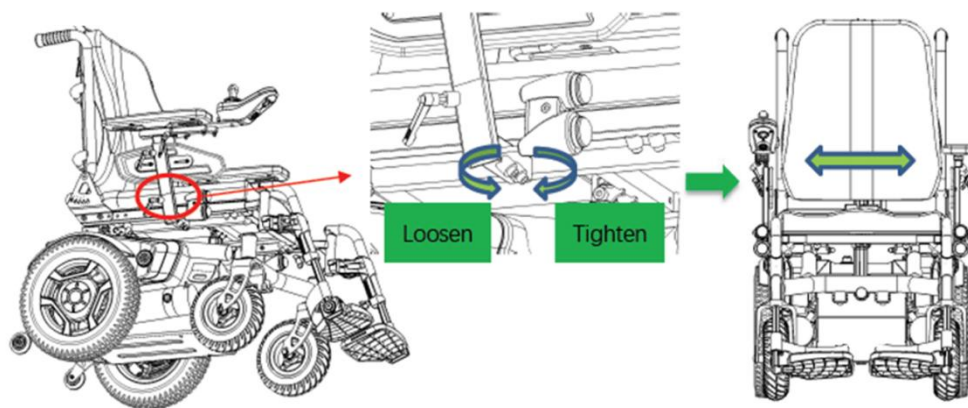

Adjusting the Height of the Armrests


1. Turn the two screws counterclockwise.

2. Loosen the armrest height adjustment tubes.

3. Move the armrest up or down to adjust it to the preferred height.


4. Turn the screws clockwise to tighten the armrest height adjustment tube.

5. Repeat steps 1–4 for the armrest on the other side.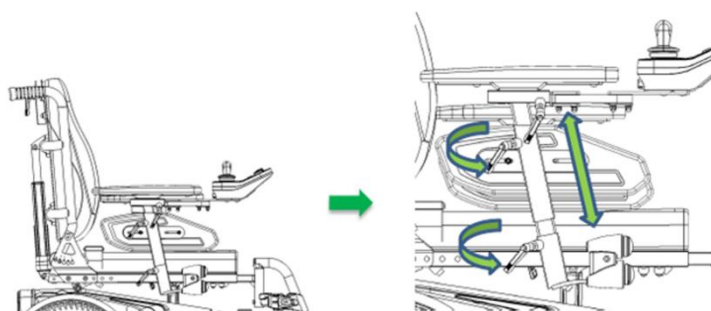


ADJUSTING THE FOOTRESTS

1. Using the tools provided with the wheelchair, turn the two screws counterclockwise.
2. Loosen the footrest adjustment tube.
3. Move the footrest outward or inward to adjust its position.
4. Repeat steps 1–3 for the footrest on the other side.
5. Check that both footrests are in the desired position.
6. Turn the screws clockwise to tighten the footrest adjustment tube.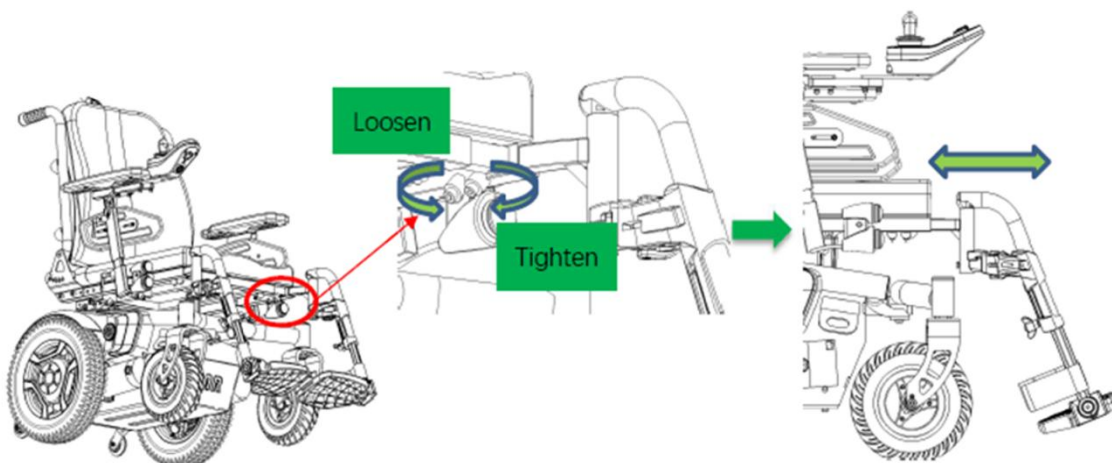


REMOVING THE FOOTRESTS

1. Lift the footplate.
2. Pull the handle outward to release the locking mechanism.
3. Rotate the footrest outward.
4. Release the handle and pull the footrest upward to remove it.
5. Repeat steps 1–4 for the footrest on the other side.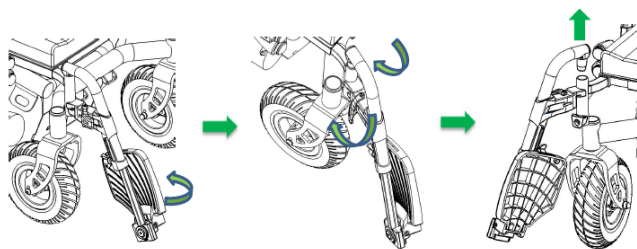

ADJUSTING THE HEIGHT OF THE FOOTPLATES

1. Turn the adjustment knob counterclockwise.
2. Loosen the footplate height adjustment tubes.
3. Move the footplate up or down to set it to the preferred height.
4. Turn the adjustment knob clockwise to tighten the footplate height adjustment tube.
5. Repeat steps 1–4 for the footplate on the other side.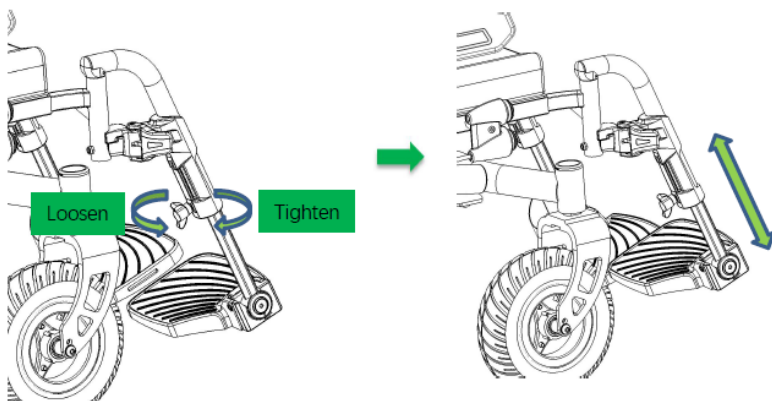

ADJUSTING THE BACKREST


Adjustment using a strap and spring-loaded locking pins

Pull the backrest adjustment strap.

Adjust the backrest angle to the preferred position.

Align the spring-loaded locking pins with the adjustment holes on both sides.

Release the strap and check that the backrest is securely locked.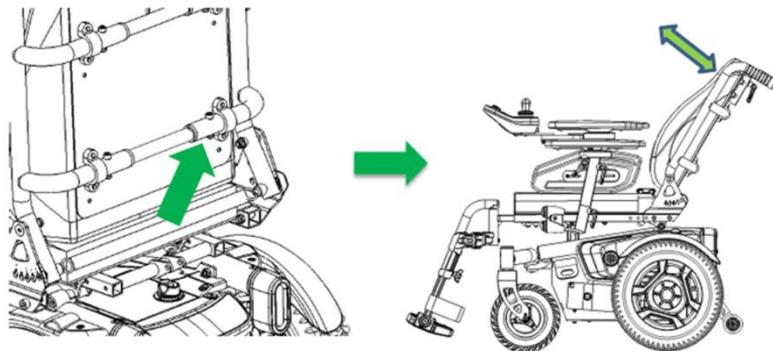

ADJUSTING THE SEAT TILT


Manual seat tilt adjustment

Remove the quick-release pin from the adjustment tube.

Raise or lower the seat to the preferred position.

Align the outer adjustment tube hole with the inner tube hole.

Insert the quick-release pin back into the adjustment tube to lock the seat position.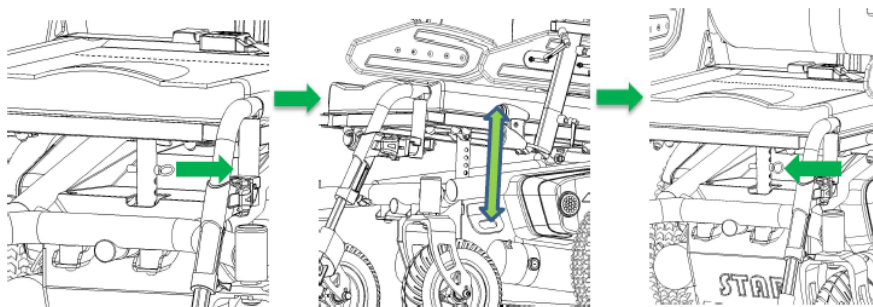

BATTERY

WARNING

If the battery symbol flashes on the charge indicator, it means the wheelchair has detected a fault. In this case, follow the procedure below:

Release the joystick, then turn the wheelchair off and on again. If this does not clear the fault, proceed to step 2.

Turn off the wheelchair and charge the battery fully. If the problem persists, proceed to step 3.

Count the number of flashes of the battery symbol. The symbol will flash a specific number of times before a longer pause. Note the number of flashes between pauses and provide this information to your dealer.

Error codes indicated by the flashing LED are listed below for reference.

ERROR DESCRIPTIONS INDICATED BY LED FLASHING

The number represents the number of LED flashes.

1: **Low battery level** – The batteries are running low.

Charge the batteries.

2: **Low battery error** – The batteries are completely discharged.

Charge the batteries.

Check the battery, connections, and cables.

3: **High battery voltage error** – The battery voltage is too high. This can occur after overcharging or when driving downhill for an extended period.

If descending a slope, reduce speed to limit regenerative charging.

- 4: **Overcurrent or controller overheating** – The motor has been drawing excessive current for a prolonged period. The wheelchair may have stopped. Turn off the controller, wait a few minutes, and turn it back on. The motor may be damaged. Check the motor, its connections, and cables.
- 5: **Parking brake error** – The parking brake release switch is active, or the parking brake is faulty. Check the parking brake, its connections, and cables. Ensure all switches are in their correct positions.
- 6: **Drive lockout** – The stop function, charge restriction, or OONAPU condition (Out Of Neutral At Power Up) is active. Disconnect the battery charger. Ensure the joystick is in the neutral position when turning on the controller. The joystick may require recalibration.
- 7: **Speed potentiometer error** – The joystick, speed limiter, SRW, or their cables may be damaged. Check the joystick, speed potentiometer, and their connections and cables.
- 8: **Motor voltage error** – The motor or its cables may be damaged. Check the motor and its connections and cables.
- 9: **Other error** – The controller may have an internal fault. Check all connections and cables.

BATTERY CHARGING

Before charging the battery, read the charging instructions in this manual and those provided with the charger. The battery charge indicator on the control console shows the approximate charge level using colors:

Green – The batteries are fully charged.

Yellow – The battery is gradually discharging.

Red – Charge the battery immediately.


For the most accurate reading, check the charge indicator while driving at full speed on a dry and flat surface.

FIRST CHARGE

Your electric wheelchair requires two 12-volt deep-cycle batteries, which are sealed and maintenance-free. The batteries are charged using the external charging system included in the package. Charge the batteries for at least **8–14 hours before the first use** of the wheelchair. To ensure long battery life, always keep them fully charged.

CHARGING THE WHEELCHAIR

Charging is done using the charging ports located on the control console. Plug the charger into the charging port on the control console. Connect the charger to a power outlet. Once fully charged, disconnect the charger from the wheelchair.

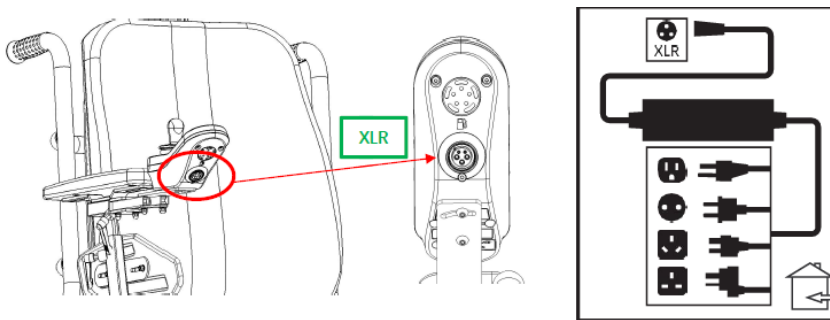OPERATING TEMPERATURES

Your electric wheelchair operates best within a temperature range of **-10°C to 50°C (14°F to 122°F)**. In extremely cold conditions, the batteries may freeze, depending on their charge level, frequency of use, and composition. In extremely hot conditions, the wheelchair may move more slowly. This is due to a built-in safety mechanism that prevents damage to the gearbox and other electronic components in case of overheating.

BATTERY OPERATION AND CHARGING

If you use your wheelchair daily:

Charge the batteries after each use to ensure the wheelchair is ready for the next day.

Recommended charging time: 8–14 hours after each day of use.

Additional charging: After the charging cycle is complete, leaving the wheelchair plugged in for an additional **4 hours** can help maximize battery life.

If you use your wheelchair once a week or less:

Charge the batteries at least **once a week for a minimum of 24 hours**.

BATTERY CHARGING

Note – Keep the batteries fully charged and avoid deep discharges. Detailed charging instructions can be found in the manual provided with the charger.

We recommend charging the batteries for at least **48 hours once a month** to improve their performance and lifespan.

Charger LED indicators:

Red light on charger – Charging in progress.

Green light on charger – Batteries are **95% charged**, and the charger is switching to **maintenance mode**.

Note – The wheelchair has a **lock function during charging**.

While charging, the wheelchair will **not operate**, and the battery indicator will be inactive.

SAFETY WARNINGS

Do not remove grounding pins – This may cause an electrical hazard. If necessary, use an approved 3-prong adapter to connect the charger to a 2-prong outlet.

Never use an extension cord to connect the charger – Always plug the charger directly into a properly wired electrical outlet.

Keep children away from the wheelchair during charging – We recommend **not charging the batteries while the wheelchair is in use**.

Explosive gases may be released during charging – Keep the wheelchair and charger away from **fire and sparks**.

Ensure proper ventilation during charging.

Charge the batteries only with the original charger – Do not use car chargers.

Inspect the charger before each use – Before charging, check the charger, cables, and connectors for damage.

If you notice any damage, **contact an authorized provider**.

Do not attempt to open the charger casing – If the charger is not functioning properly, contact an authorized provider.

The charger casing may become warm during charging – Avoid direct skin contact. Do not place it on **heat-sensitive surfaces**.

If the charger is **not designed for outdoor use**, do not expose it to extreme weather conditions. If it has been exposed to low temperatures or moisture, allow it to **adjust to indoor conditions before use**.

MAINTENANCE AND CARE

Your electric wheelchair requires **minimal maintenance and care**. If you are unsure about performing the maintenance tasks below, you can schedule an **inspection and service with an authorized provider**.

The following components require **regular checks, care, and maintenance**:

Tires

Regularly inspect the wheelchair's tires for **damage or wear**.

Wheel Replacement

Your wheelchair is equipped with **pneumatic tires**. If a tire is damaged or worn out, the **entire wheel should be replaced**.

Contact an **authorized provider** for replacement parts and information.

WARNING

The wheelchair wheels should only be serviced or replaced by a **qualified technician**.

Before starting the replacement process, ensure that the wheelchair is **turned off** and is **not in freewheel mode**.

EXTERNAL SURFACES

Components such as **bumpers, tires, and trims** can occasionally be treated with a **special rubber or vinyl care product**.

CLEANING AND DISINFECTION

Use a **damp cloth** and a **mild, non-abrasive cleaner** to clean the plastic and metal parts of the wheelchair.

Avoid using products that may **scratch the surface** of the wheelchair.

If disinfection is necessary, use an **approved disinfectant**. Ensure that the product is **safe for the wheelchair's surfaces** before application.

Warning – Before using any cleaning or disinfecting agent, read the **safety instructions**.

Failure to follow these guidelines may cause **skin irritation** or **accelerate wear on the wheelchair's upholstery and trims**.

BATTERY TERMINAL CONNECTIONS

Regularly check that the **battery terminals** are **tightened properly** and **free from corrosion**.

WIRING HARNESSES

Regularly **inspect all wiring connections**.

Check the **wire insulation**, including the **charger cable**, for signs of **wear or damage**.

If you find any **damaged wires, plugs, or insulation**, contact an **authorized service provider** for **repair or replacement** before using the wheelchair again.

DAILY CHECKS

Joystick Inspection

Check the joystick with the **power turned off**.

Ensure that it is **not bent or damaged** and that it **returns to the neutral position** when released.

Do not attempt to repair the joystick yourself. If any issues arise, contact an **authorized service provider**.

Control Console Cable Inspection

Visually inspect the **console cable** to ensure it is **not frayed, cut, or has exposed wires**.

If there are any issues, contact an **authorized service provider**.

Tire Condition Check

Inspect the tires for **any visible damage**.

Armrest Check

Ensure the **armrests** have no **loose parts, cracks, or damage**.

If any issues arise, contact an **authorized service provider**.

Brake Function Test

Perform the test on a **flat surface** with **at least 1 meter (3 feet) of free space** around the wheelchair.

WEEKLY CHECKS

Controller and Charger Connectors

Inspect the connectors for **corrosion**.

If necessary, contact an **authorized service provider**.

Pneumatic Tire Pressure

Check the **tire pressure**.

If a tire **does not hold air**, contact an **authorized service provider** to replace the inner tube.

MONTHLY CHECKS

Anti-Tip Wheels

Ensure the **anti-tip wheels** do not touch the ground **while the wheelchair is in use**.

Check for **excessive wear** and **replace if necessary**.

Tire Wear

Inspect the tires for **wear and tear**.

If necessary, contact an **authorized service provider** for a **replacement**.

General Cleaning

Remove any accumulated **dirt, dust, hair, food, or liquid spills** from the wheelchair.

ANNUAL SERVICE CHECK

Take your wheelchair to an authorized service provider for an annual inspection, especially if you use it daily.

Regular **maintenance ensures optimal performance** and prevents future issues.

AXLE BEARINGS AND MOTOR/GEARBOX ASSEMBLY

These components are **factory-lubricated and sealed**.

No additional lubrication is required.

CONSOLE, CHARGER, AND ELECTRONICS

Avoid moisture in these areas.

If any **electronic components are exposed to moisture**, ensure they are **completely dry** before using the wheelchair.

REPLACING SELF-LOCKING NYLON INSERT NUTS

Any **self-locking nylon insert nut** that has been removed during **maintenance, assembly, or disassembly** of the wheelchair **must be replaced with a new one**.

Do not reuse old nylon nuts, as this can damage the **nylon insert** and **weaken the connection**.

New nuts can be purchased from **local hardware stores** or an **authorized supplier**.

WARNING

If a "serious incident" related to the product occurs, which directly or indirectly has resulted, could have resulted, or may result in any of the following events:

- a) **death** of the patient, user, or another person, or
- b) **temporary or permanent deterioration** of the health condition of the patient, user, or another person, or
- c) **serious risk to public health**,

this **serious incident** must be reported to the **manufacturer** and the **competent authority** of the EU Member State where the user or patient resides.

For **Poland**, the competent authority is the **Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products**.

WARNING

In case of **pain, allergic reactions, or other concerning and unclear symptoms** related to the use of this medical device, consult a **healthcare professional**.

WARNING: It is strictly prohibited to use the product in a manner other than its intended purpose.

STORAGE OF THE WHEELCHAIR

If you plan **not to use the wheelchair for an extended period**, follow these storage recommendations:

Fully charge the batteries before storing.

Remove the battery from the wheelchair.

Store the wheelchair in a **warm, dry place**.

Avoid storing the wheelchair in locations **exposed to extreme temperatures**.

Recommended storage temperature: -10°C (14°F) to 50°C (122°F).

Recommended atmospheric pressure: 86 kPa to 106 kPa.

Temperature Warnings:

High temperatures may **soften the tires**.

Very low temperatures may cause **tire cracking** and increase the risk of **frostbite** for the user.

Recommended storage humidity: 30% – 70%.

☐ **Warning: High humidity** may **weaken the wheelchair's frame structure**.

Regular **deep discharging, infrequent charging, storage in extreme temperatures, or storing without a full charge** may cause **permanent battery damage**, leading to **irregular operation and reduced lifespan**.

During long-term storage, it is **recommended to periodically charge the battery** to maintain its efficiency.

To **prevent flat spots on the tires**, place **several boards under the wheelchair frame to elevate it off the ground** and relieve pressure on the wheels.

DISPOSAL OF THE WHEELCHAIR

The wheelchair must be **disposed of in accordance with local and national waste regulations**.

Contact your **local waste disposal facility** or an **authorized supplier** for information on the correct disposal of **packaging materials, metal frame components, plastic parts, electronics, batteries, neoprene, silicone, and polyurethane materials**.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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